ACP-25 (Model BR-25ST)

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
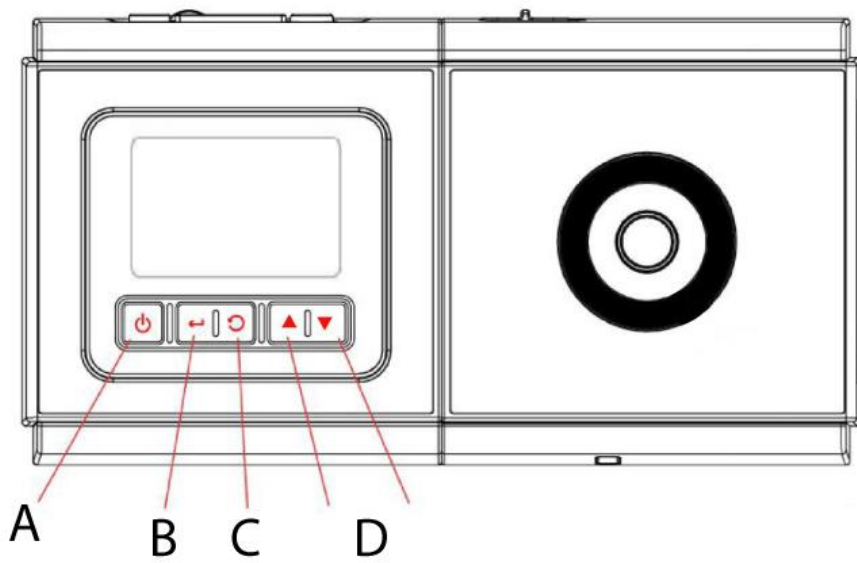
ACP-25 (Model BR-25ST)

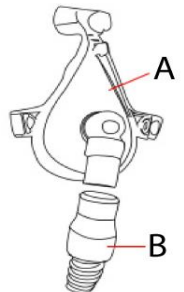
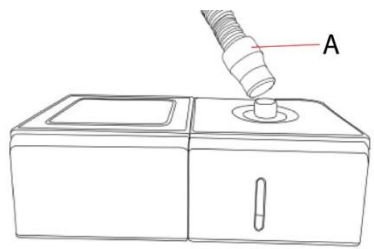
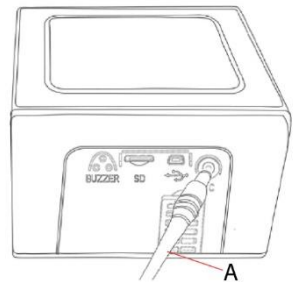
Non-invasive ventilator for use during sleep - Quick guide to operation

1. Description of the button functions

Name	Functional description
A. Power / Standby 	A short keystroke starts operation. A long keystroke ends operation (as soon as the device is switched on, it automatically goes into standby mode).
B. Select 	Selection and confirmation of menu items
C. Return 	Back to the top menu or back to the main interface
D. 	Used to move the cursor and change setting items.

2. Mounting the devices

<p>a) Connect the end of the tube to the assembled mask.</p>	<p>A. Mask B. Hose</p> 
<p>b) Connect the other end of the hose to the air outlet of the water tank.</p>	<p>A. Hose</p> 
<p>c) Connect the plug of the power adapter to the device.</p>	<p>A. Power cord</p> 

3. Start and stop the operation

Connect the device to the power supply, and the device will directly switch to the main user interface (standby status).

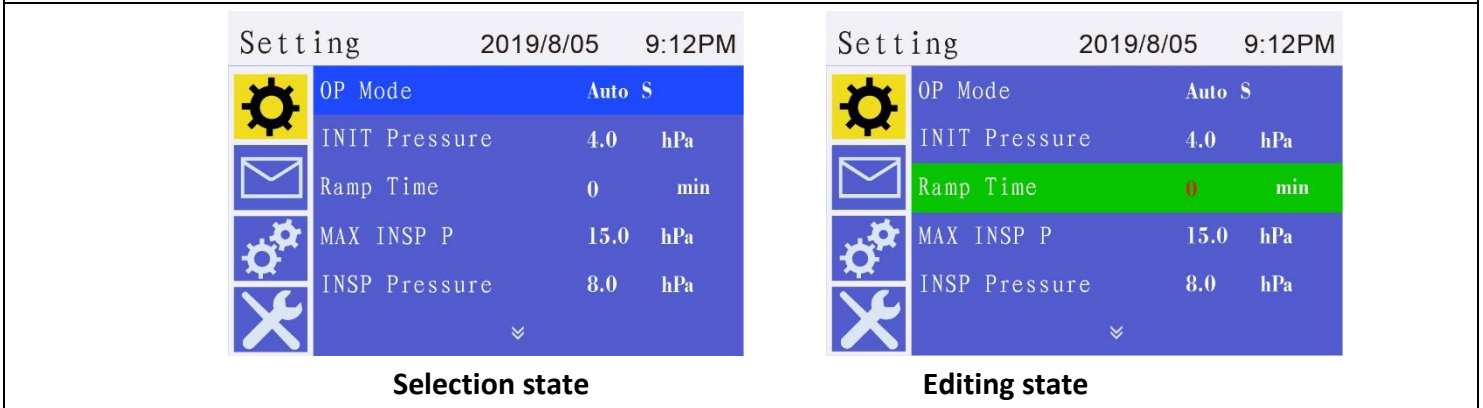
Press the [Power/Standby ] button to start operation, and press and hold the [Power/Standby ] button to stop operation.

4. Setting of the operating parameters

In standby mode, press the [Power/Standby ] and [Select ] buttons for more than 3 seconds to enter the device's setting interface.

In the blue state, various parameters can be selected. The green state is the editing state where the parameters can be changed.

5. Differences between the models

The device can operate in up to 6 modes, namely CPAP mode, AUTO CPAP mode, S mode, AUTO S mode, T mode and S/T mode. The details are as follows:

Model number	Operating mode	Maximum pressure	Remarks
BR-25ST	CPAP, Auto CPAP, S, Auto S, T, S/T	25hPa	Double level

Note: Before using the device, please read the entire operating instructions carefully to ensure proper operation of the device.

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