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## Antidecubitus mattress ADM-130 - instruction manual

### Overlay specification:

- Dimensions: 200 x 90 x 7 cm
- 130 bubble cells with foldable ends
- load capacity: 120 - 130 kg
- Weight: 2 kg
- Medical PVC
- Accessories: repair kit
- Color: Beige
- PVC thickness: 0,3 mm
- Model: HF6001

### Pump specification (compressor):

- Maximum air output:  $\geq 5$  liters/min.
- Power consumption: 7W
- Dimensions: 29 x 12 x 10 cm
- Pressure range: 50 - 100 mmHg
- Loudness: 20 dB
- Weight: 1,2 kg
- Cycle time: 6 min
- Fuse: 1A
- Model: HF6P01

### Note for users

#### DANGER – Instructions to avoid electric shock:

1. Unplug this product after use.
2. Do not use it while bathing.
3. Do not place or store this product where it may fall or slide into a tub or sink.
4. Do not place or drop it into water or other liquids.
5. Do not reach for a product that has fallen into water. Disconnect it from the power supply immediately.

#### WARNING – Instructions to avoid burns, electric shock, fire or personal injury:

1. Assess patient behavior according to patient protocol and monitor the patient accordingly.
2. This system is not suitable for patients with a spinal cord injury.
3. Careful supervision is necessary when the product is used for children or near children. Electrical burns or choking accidents may occur if a child swallows small parts that may become detached from the device.
4. Use this product only for its intended purpose as described in this manual. Do not use any other mattress not recommended by the manufacturer.
5. Never operate the product if any cord or plug is damaged, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the product to your dealer or the manufacturer for inspection and repair.
6. Keep the cable away from heated surfaces.
7. Never block the air openings of the pump or place it on a soft surface, such as a bed or couch, where the openings could be blocked. Keep the air openings free of lint, hair and other similar particles.
8. Never sink or insert any objects into openings or hoses.
9. Do not make any modifications to the device without the manufacturer's approval.
10. Avoid long pieces of tubing at the head of the bed. It could lead to strangulation.


**CAUTION** – If electromagnetic interference from a cell phone occurs, please increase the distance between the devices or turn off your cell phone.

### Product description

The anti-decubitus mattress is a nursing device for the prevention and relief of decubitus. The anti-decubitus mattress mainly consists of a pump, an inflatable mattress and a hose. The product is reusable and comes into direct contact with the patient's skin. Therefore, it can be classified as a non-invasive device intended for short-term use (less than 30 days). The product is supplied in non-sterile condition and does not require sterilization before use. The lifetime of the product is 1 year.

The inflatable mattress is made of PVC and PVC-Nylon fabric. The hose is also made of PVC. The pump consists of various electronic components and a plastic case.

For product layout see figure 1 below:

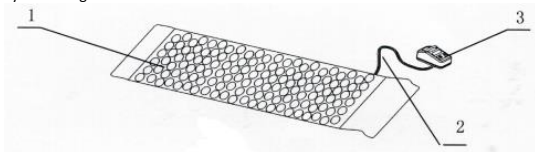1. Inflatable overlay – 2. Hose – 3. Pump

### Operating principles and mode of action

The inflatable overlay consists of spherical air chambers. After being inflated by the pump, the air chambers maintain a certain air pressure and increase the contact area between the patient's body and the mattress. The air chambers are arranged to alternately take in and release air to change the pressure point of the body to maintain normal blood circulation.

#### Predefined usage

An anti-decubitus mattress is used to prevent and alleviate bedsores in hospital, family or medical settings.

#### Instruction manual

##### 1. Preparation

- 1.1 First check the bed: there should be no sharp convex objects on it.
- 1.2 Then place the inflatable overlay on the bed; the head part should not point downwards.
- 1.3 The air intake should be positioned at the bottom of the bed.
- 1.4 Connect the pump to the support with the provided hoses and place the pump on a flat surface or hang it on the end of the bed with the provided hooks.

##### 2. Filling

- 2.1 Connect the power plug of the compressor to a nearby power socket.
- 2.2 Operate the power switch to start the inflation process.

##### 3. Mattress Hardness

- 3.1 The hardness of the mattress is determined by the inflowing air, which can be regulated by adjusting the rotary control located on the compressor (clockwise rotation increases the air volume, counterclockwise rotation decreases it).

#### Safety instructions

1. In order to ensure proper operation of the air cushions, the air output should never be blocked.
2. Handle all components with special care to avoid damage.
3. Do not load the mattress with more than 130 kg.
4. Use only cleaning substances with a CE marking and adhere to the processing times specified by the manufacturer.
5. Use the pump only in a dry and clean place - in addition, make sure that the environment is free of dust.
6. When the mattress is not in use, turn off the pump by toggling the power switch, remove the pressure hose to allow the air to escape, and roll up the mattress.
7. To reduce the wear of the mattress, clean it carefully after each use and store it in a properly ventilated room.
8. This product does not generate strong magnetic waves, but to ensure proper operation of other medical equipment, it should not be used in conjunction with other highly sensitive instruments and devices.
9. After the end of its lifespan, the product should be recycled.

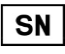
**Warning:** No modifications may be made to this device!

Further specifications	
Ventilation hose	Material: PVC, Length: 1200 mm
Operational environment	Temperature: 5 ° C – 40 ° C Relative air humidity: ≤ 80 % Atmospheric pressure: 86 kPa ~ 106 kPa
Power Supply	AC 230 V
Nominal frequency	50 Hz
Mode of operation	Continuous operation
Overvoltage category	1
Load degree	2
Protection degree	Type BF
Waterproofness	IPX0, no protection

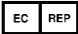
#### Storage and transportation

Store the anti-decubitus mattress fully packed and protected from sun and rain. Transport the item only under the above conditions.

#### Symbols

	BF type application part		Serial number
---	--------------------------	---	---------------

	Recycling according to local laws		Date of manufacture
	Observe the instructions for use		Manufacturer
	Caution		European Union representative
	Batch code		Keep dry
	Fragile, handle with care		Keep upright during transport
	Expiration date		CE marking

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### Directives and manufacturer's specifications - Electromagnetic emissions

The anti-decubitus mattress is intended for use in the electromagnetic environment specified below. The customer or the user of the anti-decubitus mattress should ensure that it is used exclusively in such an environment.

IEC-STANDARD	Conformity	Electromagnetic environment - instruction
CISPR 11	Group 1	The anti-decubitus mattress uses RF energy only for its internal function. Therefore, its RF emissions are very low and unlikely to cause interference with nearby electronic equipment.
CISPR 11	Class B	The anti-decubitus mattress is suitable for use in all facilities, including private residences and those directly connected to a low-voltage network that supplies buildings used for residential purposes.

### Guidelines and manufacturer's specifications - Electromagnetic immunity

The anti-decubitus mattress is intended for use in the electromagnetic environment specified below. The customer or the user of the anti-decubitus mattress should ensure that it is used exclusively in such an environment.

Interference immunity test	IEC60601 test level	Conformity level	Electromagnetic environment - Instruction
Electrostatic discharge (ESE) IEC 61000-4-2	±6 kV Contact discharge ±8 kV Air discharge	±6 kV Contact discharge ±8 kV Air discharge	The floors should be made of wood, concrete or ceramic tiles. If the floor is covered with synthetic material, the relative humidity should be at least 30%.
Voltage peaks & discharges IEC 61000-4-4	±2 kV for Power lines	±2 kV not applicable for power lines	Correspond to a typical business or hospital environment.
Overvoltage IEC 61000-4-5	± 1kV Push-pull voltage ± 2kV Common mode voltage	± 1kV Push-pull voltage ± 2kV Common mode voltage	
Voltage dips, short interruptions and voltage fluctuations on the input lines of the power supply IEC 61000-4-11	<5% UT (>95% dip of UT) for 0.5 cycles 40% UT (>60% UT collapse) for 5 cycles 70% UT (>95% UT collapse) for 0.5 cycles <5% UT (>95% dip of UT) for 5 seconds	<5% UT (>95% dip of UT) for 0.5 cycles 40% UT (>60% UT collapse) for 5 cycles 70% UT (>95% UT collapse) for 0.5 cycles <5% UT (>95% dip of UT) for 5 seconds	The quality of the power supply should be that of a typical business or hospital environment. If the user of the anti-decubitus mattress requires continuous operation during power interruptions, it is recommended that the anti-decubitus mattress be powered by an interruption-free power supply or a battery.
Line frequency (50 Hz) Magnetic field IEC 61000-4-8	3A/m	3A/m	The quality of the mains voltage should correspond to that of a typical business or hospital environment.

Note 1: UT is the AC line voltage before the test level is applied.

### Guidelines and manufacturer's specifications - Electromagnetic immunity

The anti-decubitus mattress is intended for use in the electromagnetic environment specified below. The customer or the user of the anti-decubitus mattress should ensure that it is used exclusively in such an environment.

Interference immunity test	IEC60601 Testing level	Conformity level	Electromagnetic environment - Instruction
Wired RF IEC61000-4-6	3 Vrms 150 KHz to 80 MHz 3 V/m	3 Veff	Portable and mobile RF communication devices should not be used closer to any part of the anti-decubitus mattress, including cables, than the recommended distance calculated using the equation applicable to the frequency of the transmitter. Recommended distance= $(3.5/V1)\sqrt{P}$ $d=(3.5/E1)\sqrt{P}$ P80 MHz to 800 MHz $d=(7/E1)\sqrt{P}$ 800 MHz to 2.5 GHz Where P is the maximum output power of the transmitter in watts (W) as specified by the transmitter manufacturer and d is the recommended distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be below the compliance level in each frequency range. Interference may occur in the vicinity of devices marked with the following symbol: 
Radiated RF IEC61000-4-3	80 MHz to 2.5 GHz	3V/m	

NOTE 1: For 80MHz and 800MHz, the higher frequency range applies..


NOTE 2. these guidelines may not be applicable in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

Field strengths from fixed transmitters, such as base stations for radiotelephones (cellular/cordless phones) and land mobile radios, amateur radio, AM and FM radio and television broadcasts, cannot theoretically be accurately predicted. To assess the electromagnetic environment of fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength at the location where the pressure ulcer mattress is used exceeds the RF compliance level specified above, the pressure ulcer mattress should be observed to ensure proper operation. If abnormal behavior is observed, additional measures may be required, such as realignment or relocation of the anti-decubitus mattress. The frequency range from 150 KHz to 80 MHz, the field strengths should be less than 3 V/m.

**Importeur/ Importer / Importador/ Importatore / Importör:**  
Novidion GmbH, Fuggerstr. 30, 51149 Köln, Germany

**Foshan Hongfeng Co., Ltd**  
No.4-2 Leqiang Road, Leping Sanshui, Foshan, 528100, Guangdong, China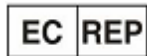

**MedNet EC-REP GmbH**

Borkstrasse 10  
48163 Muenster  
Germany

**Bei Fragen wenden Sie sich bitte an /** If you have any questions, please contact / Si tiene alguna pregunta, póngase en contacto con nosotros / Si vous avez une question s'il vous plaît contacter / Se si dispone di una domanda si prega di contattare / Als u vragen heeft, kunt u contact opnemen met / W razie jakichkolwiek pytań prosimy o kontakt / Om du har några frågor, vänligen kontakta:

Novidion GmbH  
Fuggerstr. 30  
51149 Köln  
Tel.: +49 (0) 2203 9885200  
Fax: +49 (0) 2203 9885206  
info@pulox.de, www.pulox.de  
WEEE: DE24355330

**Sollten Sie beim Lesen dieser Anleitung Probleme haben, so können Sie diese auch auf unserer Website downloaden:** / In case you have any problems reading this manual, you can download it from our website: / En caso de que tenga algún problema para leer este manual, puede descargarlo de nuestro sitio web: / Si vous rencontrez des problèmes pour lire ce manuel, vous pouvez le télécharger depuis notre site Web: / In caso di problemi nella lettura di questo manuale, è possibile scaricarlo dal nostro sito Web / Als u problemen hebt met het lezen van deze instructies, kunt u ze ook downloaden van onze website / W przypadku problemów z odczytaniem tych instrukcji, można je również pobrać z naszej strony internetowej / Om du har problem med att läsa dessa instruktioner kan du också ladda ner dem från vår webbplats:

**<https://www.pulox.de/Anleitungen-Datenblaetter-Retouren>**