ChokeVAC

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Dear Customer,

we are pleased that you have decided to purchase a product from our range. Please read these operating instructions carefully and carefully and keep them accessible for other users. Observe the application instructions and notes.

With kind regards
the Novidion team

Choke Suction Device "ChokeVAC" Emergency choking device Instruction manual - In accordance with Regulation (EU) 2017/745 Annex II and III

1. Intended use

The function of the "ChokeVAC" emergency asphyxiation device is to reduce the risk of asphyxiation in the event of airway obstruction. The device is used in emergency situations, especially when someone is in danger of choking. It is used through appropriate pressure and skillful technique to clear airway blockages and restore airflow to the lungs, saving the person's life. This device provides additional support during an emergency choking treatment to ensure the airway remains clear and the victim's breathing is preserved.

The device is applicable to individuals 1 year of age and older (or at least 10 kg body weight) and should only be operated by individuals who are in full possession of their physical, sensory and mental faculties.

The product is suitable for use in private homes and medical facilities.

2. Contraindications

- Do not use the product if the patient does not show signs of choking and can breathe on their own.
- Patients with severe damage or infections in the larynx area
- Patients with significant vascular disease or a history of neck surgery
- Pregnant women and patients with valvular heart disease may only be treated on medical advice

3. Safety instructions

Please read the safety instructions carefully before using the equipment. Operators do not need professional training, but should use this product only after fully understanding the requirements in this manual. To avoid damage from improper use, use this product according to its intended use.

1. If there is any visible damage to the device or the mask used, please do not use the product and contact the manufacturer or distributor.
2. Not suitable for children under 28 days old.
3. Use of the device cannot guarantee resuscitation of a patient.

Therefore, use at your own risk. The manufacturer is not responsible for any injury and/or death resulting from its use.

4. The product may present a residual risk of injury to the patient.
5. Use the product only in accordance with the operating instructions to minimize the risk of non-excludable injuries (such as bruising of the face or coughing after removal of the airway blockage). Failure to follow the instructions may result in the death of the patient.
6. Do not use the device for any other purpose not described in these operating instructions, as this could result in risks for patients and users.
7. This product is not a toy. Store the product out of the reach of children.
8. If 2-3 attempts to use the device are unsuccessful and the blockage in the respiratory tract persists, please immediately consult an emergency physician or other medical professionals. Failure to do so may result in the patient's death.
9. In case of use of the product for different persons, cross-infection could occur.
10. All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is established.
11. This product is disposable and cannot be used more than once.

Caution: Repeated use may cause harm to the patient due to infection.

4. Product features

Product parameters:

Product name:

ChokeVAC Choke Suction Device

Model: TW8648A

Basic UDI-DI: 697426279CSDTY

Classification: Medical device class I

Performance features:

A. Easy to use and suitable for non-

professionals

B. Handy, lightweight and mobile use

C. Effective help in acute choking situations

Materials included:

- PC (Polycarbonate)
- TPE (Thermoplastic Elastomer)
- SS (Stainless Steel)
- PVC (Polyvinyl chloride)

Information on other substances:

- Contains bisphenol A
- Latex free
- Phthalate free

5. Accessories

Soft face masks that are gentle on the skin and do not create the risk of head trauma. Included are:

1x Mask for adults (large)

1x Mask for adults (small)

1x Mask for children


Product model	Face mask model (Air Cushion Masks - without Check Valve)	length (mm)	width (mm)	height (mm)
TW8648A	0#-Neonate (with connector)	65±5	50±5	45±5
	1#-Infant (with connector)	80±5	65±5	50±5

	2#-Pediatric	95±5	75±5	65±5
	3#-Adult Small	115±5	90±5	90±5
	4#-Adult Medium	127±5	100±5	80±5
	5#-Adult Large	135±5	110±5	80±5
	6#-Adult Extra Large	145±5	115±5	90±5

6. Structure of the product

The product mainly consists of a handle, plunger, vent valve and face mask. It is a non-electric, non-invasive device for removing airway obstructions. It is equipped with a special valve that prevents air from escaping through the mask. The special valve prevents air from pushing down food or objects. It can quickly and effectively remove the airway obstruction when the asphyxiating patient cannot breathe and save the life of the asphyxia patient.

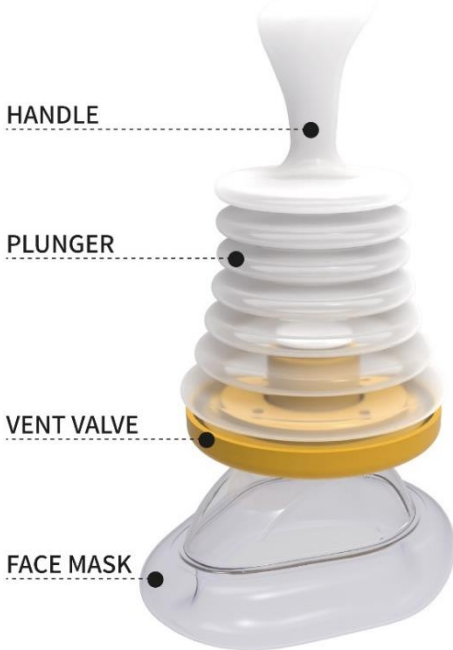The necessary negative pressure is generated by the pressure piece (plunger). The vent valve, through which the air is released from the main body, ensures that the necessary suction can be generated.


7. Application in emergency situations

1. Place: Place the mask over the nose and mouth of the affected person. Hold the mask firmly with your weak hand to seal the mask as best as possible.
2. Push: Press the plunger down with your strong hand. This allows all the air to escape through the one-way valve system and prevents an obstruction in the airway from being pushed in further.
3. Pull: Pull the plunger back while continuing to hold the mask in place. This creates the suction to remove the obstruction. If the obstruction cannot be removed on the first attempt, continue to push and pull

without removing the device from the patient.

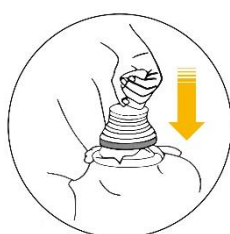1 PLACE

2 PUSH

3 PULL

8. Storage conditions

Storage temperature: $-40^{\circ}\text{C} \sim +60^{\circ}\text{C}$ (the relative humidity when storing the product must not be higher than 85 %)

Store the product in a safe, dry place that is easily accessible in the event of an emergency. Do not expose the device to direct sunlight and do not store it in environments where extreme heat or cold could affect the functionality of the device. An ideal storage location would be a kitchen cabinet.

Shelf life: Three years from the end of production.

9. Symbols

Symbol	Meaning	Symbol	Meaning
	Attention!		Attention! Please refer to the accompanying document (user manual)
	Observe instructions for use		Unique device identification
	Medical device		Serial number
	Recyclable		Keep upright during transportation
	Batch number		Expiry date
	Store in a dry place		Protect from direct sunlight
	Translation		Outer packaging
	Importer		Catalog number
	Date of manufacture, made in China		manufacturer
	European representative		Latex-free
	Non-sterile		For single use only
	Phthalate free		Fragile, handle with care
	This product complies with the Medical Device Directive (EU) 2017/745 (MDR)		

Novidion GmbH
 Fuggerstr. 30 • 51149 Cologne • Allemagne
 Tel.: +49 (0) 2203 / 9885 200
 Fax: +49 (0) 2203 / 9885 206
 www.pulox.de • Mail: info@pulox.de

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Xiamen Dahton Medical Co., Ltd.
SUNGO Europe B.V.