NA-53

Nasensauger

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Nasal aspirator

- Operating instructions

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- Mode d'emploi

Aspiratore nasale

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Aspirador nasal

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Neusafzuiger

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Aspirator do nosa

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Nasal aspirator

- Bruksanvisning

www.pulox.de

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Operating instructions

Dear Customer,

Thank you for purchasing a product from our range. Please read the operating instructions carefully before use.

Kind regards,

Your Novidion team

Intended use

The nasal aspirator is intended solely for the removal of nasal secretions in children and babies. Do not use the nasal aspirator for other purposes or on animals. Children over 8 years of age, disabled persons with limited physical or mental abilities, and persons without experience or knowledge can operate the device in a safe environment if they are supervised and understand the risks involved. Children should not play with the device and should only clean and use it under supervision.

Operate the device in accordance with the instructions in this manual.

Keep packaging material out of reach of children (risk of suffocation).

This nasal aspirator is for private use only and is not suitable for medical or commercial purposes. The nasal aspirator may only be used for the purpose for which it was designed and in the manner specified in the instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damages resulting from improper or incorrect use. Do not use the nasal aspirator if one or more of the following warnings apply.

Technical details

Type	VP-X1A
Dimensions	142 x 42 x 56mm
Weight	approx. 137g (without batteries)
Maximum suction pressure	70 kPa
Sound power level	< 50 dB
Power supply	2 x 1.5 V AA alkaline batteries
Operating conditions	Temperature: 10°C to 40°C Humidity: 30% to 85%RH Atmospheric pressure: 70kPa ~

	100kPa
Storage and transportation conditions	Temperature: -20°C - +50°C Humidity: 30% to 85%RH Atmospheric pressure: 70kPa ~ 100kPa

Technical modifications reserved.

The serial number can be found on the device or in the battery compartment.

Please note that portable and mobile RF communication systems can interfere with this device.

Notes on electromagnetic compatibility:

- The device is suitable for use in all environments listed in this manual, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic interference. This could lead to problems such as error messages or failure of the display/device.

Avoid using this device directly adjacent to or stacked with other devices, as this could result in improper operation. However, if it is necessary to use the device in this manner, both this device and the other

device should be monitored to verify that they are operating normally.

- Use of accessories other than those specified or provided by the manufacturer of this device may result in increased electromagnetic emissions or decreased electromagnetic immunity of the device, which could result in improper operation.
- Failure to observe the above points may impair performance.

Included in the delivery


Check the outer packaging of the delivery for damage and make sure that all components are enclosed:

- 1 x Nasal aspirator
- 1 x Broad silicone tip
- 1 x Slim silicone tip
- 2 x 1.5 V alkaline batteries AA
- 1 x Secretion container

Description of the device

1. Silicone tip
2. Secretion container
3. Protective cover for container
4. Front shell


5. Button for the first stage
6. Button for the second stage
7. Operating control lamp
8. Gas vent
9. Battery compartment
10. Sound outlet
11. Battery compartment cover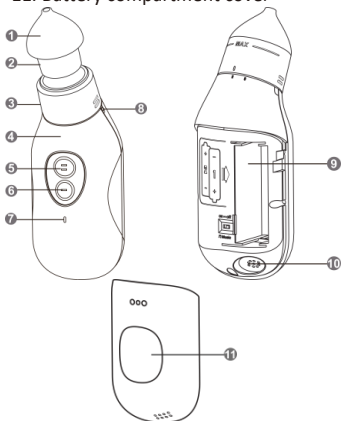

First use

Inserting the batteries:

1. Slide the battery compartment cover off.
2. Insert batteries into the compartment according to the indicated polarity.
3. Slide the cover back onto the battery compartment until it clicks into place.

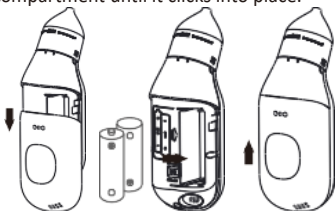Use

To begin the process, proceed as follows:

1. Check that the device is working properly and is properly closed.
2. Hold or place the child to be treated in an upright position or in a position where they are leaning back slightly. Do not use the device when the child is lying down.
3. Hold the device at an angle of 30° to 60° to your face during treatment.

You can choose between the silicone tips as needed:

Broad silicone tip


Slim silicone tip

4. The nasal aspirator only works when you press the button for the first or second suction level.
5. Stop using the product immediately if the child experiences discomfort or pain.
6. Stop using the device as soon as the nasal secretion container is filled to the "MAX" line.
7. Release the button to stop using it.
8. Clean the device after each use (see next chapter).

Cleaning and disinfection

ATTENTION: Clean the device after each use, otherwise it may become clogged and cannot ensure smooth and hygienic operation.

1. Remove the used silicone tip and the secretion container from the device (squeeze the silicone tip and pull it out with a little force together with the secretion container).

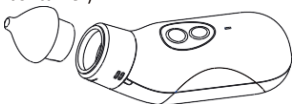2. Disconnect the parts and wash them separately.

3. The silicone tips and the secretion container can be cleaned with water or a mild soap solution.

4. Place the parts on a dry, clean, absorbent surface and allow them to dry completely.

Silicone tips and secretion containers can be sterilized with alcohol.

Clean the main unit only with a clean and dry cloth.

Do not use any caustic cleaning agents, solutions or benzenes.

The unit must not be rinsed with water or immersed in water.

Dry all parts carefully with a soft cloth. Do not reassemble any parts until they are completely dry. Store the device out of sunlight, dust and moisture.

Disposal of the device

For environmental reasons, do not dispose of the device in the household waste at the end of its service life. Dispose of the device at the appropriate local collection or recycling point in your country. If you have any questions, please contact the authority responsible for waste disposal.

Disposing of batteries:

The empty, fully discharged batteries must be disposed of via specially designated collection boxes, recycling points or the electronics trade. You are legally obliged to dispose of the batteries.

Troubleshooting

Problem	Possible causes	Solution
The suction power is reduced/non-existent.	Batteries not inserted correctly / battery has no more power.	Check that the batteries are inserted the right way / Replace the batteries
The secretion cannot be absorbed.	Dryness turns sticky secretions into hard boogers	It is recommended to instill 1-2 drops of regular saline solution into the baby's nasal cavity, gently massage it for 10 minutes, and then use a nasal aspirator to gently suction the boogers. Repeat the step until the baby's nasal cavity is clear.

If secretion leaks out of the nasal secretion container	Excessive secretion; nasal secretion container not cleaned in time	Turn the protective cover of the nasal secretion container, remove it together with the container, rinse it with clean water and wipe the device with a clean cloth.
---	--	--

Signs and symbols

The following symbols appear in these instructions and on the device:

	WARNING: This indicates a risk of injury or damage to health.
	ATTENTION: Safety notice indicating a possible risk of damage to the device/accessories.
	Follow the instructions for use
	Type BF applied part
	Disposal in accordance with the EU Directive on Waste Electrical and Electronic Equipment - WEEE
IP 22	Protected against foreign bodies $\geq 12.5\text{mm}$ and against water falling at an angle of 15 degrees.
	This product fulfills the requirements of the applicable European and

	national directives.
	Manufacturer
	Permissible temperature and humidity during storage and transportation.
	Permissible operating temperature and humidity

Warnings and safety information

WARNING

- Before use, make sure that the device and accessories show no visible damage. If you have any doubts, do not use the device and contact the customer service.
- If your child has a nasal disease, consult your doctor before use.
- Use the device only as described in the instructions. Failure to follow the warnings may result in injuries such as nosebleeds, mucosal bleeding, discomfort, or pain.
- Clean the silicone tips before each use.
- If the device is not working properly, stop using it immediately.
- If discomfort or pain occurs during operation, stop use immediately.
- Make sure that the baby does not hold its breath, close its mouth or swallow its

saliva during operation. This could create a vacuum that can damage the eardrum.

- This device is not suitable for the following groups of people: individuals with sensory (e.g. reduced sensitivity to pain) or mental problems or a lack of experience or knowledge, unless they are supervised by a person responsible for their safety or have been instructed in the use of the device.

- Keep packaging material away from children (risk of suffocation).

- Never open, modify or repair the device, otherwise proper functioning can no longer be guaranteed. Failure to follow these instructions can be dangerous and will also void the warranty.

- Do not use the device if it has been dropped, exposed to heavy moisture or has suffered any other damage. If in doubt, contact customer service or your specialist dealer.

WARNING

- Do not use the device on persons with nosebleeds, acute head, face / neck injuries, or blood clotting or blood diseases.
- Do not use excessive force when inserting the silicone tips.
- Do not use any parts that are not recommended by the manufacturer.
- Under no circumstances should liquid be allowed to enter the device. Protect the device from shocks, moisture, dirt, strong temperature fluctuations and direct sunlight. Do not drop.

Notes on handling batteries:

- Rechargeable batteries must not be used in this device!
- If the batteries are flat or you do not intend to use the device for a long period of time, remove the batteries from the device. This will prevent damage that could be caused by leakage.
- Danger of choking! Keep batteries out of the reach of children.
- Pay attention to the plus (+) and minus (-) polarity.

- If a battery has leaked, put on protective gloves and clean up the battery. If the battery fluid has come into contact with your skin or eyes, rinse the affected area with water and seek medical attention.
- Protect the batteries from excessive heat.
- Risk of explosion! Do not throw batteries into a fire.
- Use only identical or equivalent battery types.
- Always replace all batteries at the same time.
- Do not disassemble, open or crush batteries. Remove discharged batteries from the device and dispose of them safely.
- The supply terminals must not be short-circuited.

Safety Instructions / Warranty & Service IMPORTANT!

If you have any questions or require assistance, please contact our telephone support team on +49-2203-9885200 or email us at info@novidion.de.

The main unit is covered by a statutory warranty of two years.

If the main unit fails to operate properly under normal use within this two-year period, Novidion GmbH will, at its discretion, either repair or replace the product free of charge.

The product must be accompanied by proof of purchase, either a receipt or other proof that the device is within the warranty period. Novidion GmbH will bear the costs of repairing or replacing the product and returning it to you. This warranty does not apply to a product that has been damaged by improper maintenance, an accident, improper power supply or any other form of misuse. The warranty also expires if the owner repairs or modifies the product in any way.

Novidion GmbH is not liable for incidental or consequential damages in connection with this product. The warranty also excludes any liability other than that stated above. No other warranty is given.

LEGAL RIGHTS VARY FROM STATE TO STATE. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION MAY NOT APPLY TO YOU.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void your authority to operate the equipment.

CAUTION: To reduce the risk of electric shock, do not expose this device to rain or moisture.

Protect the environment! This electronic device contains recyclable materials and must not be disposed of with household waste. Please take it to a collection point for electrical and electronic equipment (ask your local authorities for details).

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Vapo Healthcare Co.,Ltd
Share Info GmbH