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www.pulox.de / info@pulox.de

Dear user:

Thank you for choosing our arm type blood pressure monitor. Before using this arm type blood pressure monitor, please read the user manual carefully and use it accordingly. Please keep the user manual properly for reference at any time. The arm type blood pressure monitor can be used in hospitals, families, schools and medical centers. It is suitable for adults, not for pediatric and neonate use.

Brief introduction of the product:

The full name of this product is arm type blood pressure monitor. It consists of the main body and the arm cuff and is suitable for human blood pressure and pulse measurement.

The blood pressure monitor adopts the intelligent pressure method, which can measure your blood pressure to a suitable value automatically according to your own blood pressure value. It can effectively reduce the discomfort caused by incorrect pressure, shorten the measurement time, and prolong the service life of the cuff at the same time.

The blood pressure monitor can be used with a power adapter socket (not included), which is easy to use.

This blood pressure monitor has a memory function of 99 groups of measuring data for two users, which can save the data separately. It can display the average reading of the latest 3 groups of

measurement results.

This blood pressure monitor has the function of blood pressure classification, which is convenient for you to judge whether your blood pressure is normal or not.

This blood pressure monitor has a voice broadcast function (optional). During the measurement and recall of the memory, instructions for operation are given by voice.

Safety Precautions

Warnings and graphic symbols in the user manual are aimed at enabling users to use the product safely and correctly, so that any damages to users and others can be avoided. Specific meanings are as follows:

	Warning: Reminds you that misuse of the product may cause personal damage, measurement failure and property damage. (Property damage includes damage to housing, property, pets and livestock.)
	Enforcement: Represents compulsive behavior, namely matters must to be observed when using the product
	Prohibition: Stands for prohibited contents, namely matters not allowed to do when using the product
	Prohibition of Disassembly
	Caution of electric current
	Caution of fire

	BF type applied part
	Refer to operation manual
	Manufacturer
	Date of manufacture
	European union representative
	Serial number
	Separate collection
	Keep dry
	Product certification
IP21	First characteristic numeral 2: Against ingress of solid foreign objects: $\geq 12.5\text{mm}$ diameter Second characteristic numeral 1: Against ingress of water with harmful effects: vertically dripping
	Medical device
	Swiss authorized representative

- Use the product only under medical supervision in case of arrhythmia, diabetes, circulatory disorders or stroke.
- Do not self-diagnose or self-treat based on the measurement results. Please follow the doctor's instructions, self-diagnosis may worsen the patient's condition.
- Over-inflation of the cuff may cause numbness in the arm or pain.
- Do not place the cuff over a wound as this may cause further injury.
- For safety reasons, store the product in a place that children cannot reach.
- Do not apply the cuff to limbs that already have intravascular access or an arterio-venous (A-V) shunt. This may cause injury to the patient due to temporary impairment of blood flow.
- Inflation of the cuff may cause temporary loss of function of a blood pressure monitor used simultaneously on the same extremity.
- Check that the operation of the blood pressure monitor does not cause prolonged impairment of the patient's blood circulation.
- Long cables and tubing may pose a strangulation hazard and should be kept out of the reach of children.

- Infants and young children must not use the product as accidents may occur.
- Only use the product to measure blood pressure, otherwise it may cause accidents.
- Please do not use chemical reagents such as thinner, alcohol or

benzine to clean this product.

- Do not bump or drop the product.
- Please be sure to use the special arm cuff of this product, otherwise accurate measurement will not be obtained.
- Do not use your cell phone near the product. There is a risk of malfunction of the product.

- Do not disassemble, repair or reconstruct the device and the cuff of the blood pressure monitor, otherwise it may result in erroneous measurements.

- Do not bend the cuff or the air hose forcibly. Blood flow could be impaired and the patient could be injured.
- If you want to remove the air hose, take out the air hose at the front part.
- Do not start the device until you have put the arm cuff on your arm.
- Since battery leakage may damage the product, please observe the following points.
- If you do not use the product for a long time (more than 3 months), please remove the batteries.
- Replace dead batteries with new ones immediately after use.
- Do not use old batteries together with new batteries.
- Insert the batteries correctly according to the positive and negative poles.
- Do not use different types of batteries together.
- Do not modify this device without the manufacturer's approval.

- Do not repair or service the blood pressure monitor while it is in use on the patient.

- Please use the special AC power supply of this product, otherwise it may cause fire or electric shock.
- Be sure to use the appropriate AC power adapter. Use a separate power outlet, otherwise it may cause a fire or electric shock.

- Do not unplug the AC regulated power supply with wet hands, as this may result in electric shock or injury.

1. Product Introduction

1.1 Intended use


The blood pressure monitor is used for measuring the blood pressure and pulse rate. The monitor can be used in hospitals, families, schools and medical centers. It is suitable for adults, not for neonate.

User Group Age: Adult - People who need to have their blood pressure measured

1.2 Contraindications


Serious arrhythmia

1.3 Device description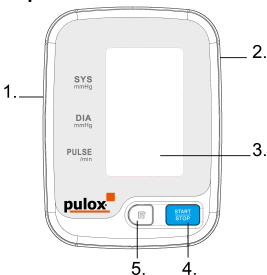


Front:

- | | |
|--------------------------------------|----------------------|
| 1. Connection of the air hose | 4. Start/Stop button |
| 2. Connection of the AC power supply | 5. Memory button |
| 3. Display | |

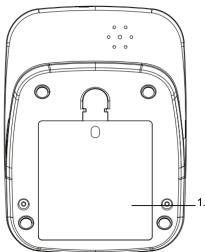Backside:
1. Battery cover

1.4 Display Screen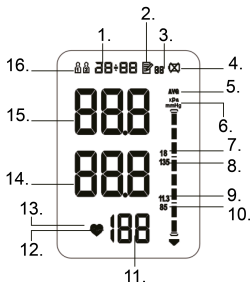

- | | |
|--------------------------------|-----------------------------|
| 1. Date/Time | 9. kPa lower limit |
| 2. Memory symbol | 10. mmHg lower limit |
| 3. Number of memory groups | 11. Pulse Rate |
| 4. Low power symbol | 12. Heart Rate |
| 5. Average symbol | 13. Heart rate fluctuations |
| 6. Blood pressure unit symbols | 14. Low pressure |
| 7. kPa upper limit | 15. High pressure |
| 8. mmHg upper limit | 16. User symbol |

1.5 Functional Description

- 1) Measurement: Oscillographic measurement;
- 2) Results show: high pressure / low pressure / pulse;
- 3) Unit Conversion: blood pressure units kPa / mmHg conversion (the default boot unit is mmHg);


- 4) Memory Group: Two sets of memory, each 99 measurement results of memory;
- 5) Clock function: Set of year, month, day, hour, minute;
- 6) Low power testing: any working-state detecting low power, LCD displays  symbol prompts low power;
- 7) Blood pressure classification indicator: blood pressure classification indicates blood pressure health, see Appendix III;
- 8) Error: See Appendix Table IV;
- 9) Over pressure protection function: pressure over 295mmHg (20ms) is automatically and quickly exhaust;
- 10) Auto power off function: No action for 2 minutes then automatic shutdown.

2. Battery / AC adapter use

This product can be used both with batteries and AC adapter (not included in delivery).

2.1 Use of batteries

- 1) Remove the battery cover in the direction of the arrow;
- 2) Put in four AA batteries;
- 3) Close the battery cover.

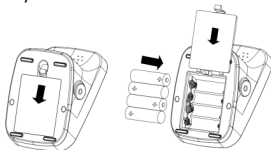Note

- Icon "": indicates the battery is about to run out. Please

replace the four batteries (same species).

- Please turn off the machine when the battery is replaced.
- If you remove the battery in the boot state, the timer is stopped.
- After replacing the battery, you may need to reset the time. For details, refer to "3.2 Setting the Time" section.
- Methods of disposal of the batteries should be handled according to local regulations on environmental protection.
- Note: battery life and charge times vary depending on usage and settings.
- Battery life: after the battery is fully charged, it can complete at least 180 times of measurement, and the battery can complete at least 300 times of charge and discharge.

2.2 AC Adapter for use

- 1) Connect the blood pressure monitor with AC adapter;
- 2) Keep the AC adapter plug into the right side of the arm type Blood Pressure Monitor AC power adapter interface.
- 3) Plug the connected AC adapter into an AC outlet.

Note

- When the device is off, cut off the power socket and adapter, then cut off the AC adapter and arm type Blood Pressure Monitor. Be sure to use a dedicated AC adapter.
- When the power supply and battery use at the same time, it does not consume battery power;
- When switching AC adapter and battery, please switch in the shutdown state, otherwise the instrument will shut down because of power outages;
- Only use the AC adapter, unplug it from the socket the set time will disappear, in order to maintain the correct display time, use

both batteries.

Note

- When the power is off, cut off the power socket then cut off the adapter and blood pressure monitor. Be sure to use a dedicated AC adapter.
- When the adapter and the battery are used at the same time, the battery does not consume power.
- When switching adapter and battery, please switch in the shut-down state.
- Only use the AC adapter, unplug it from the socket, the set time will disappear. In order to maintain the correct display time, use both batteries and adapter.

3. Initial Setting

This device can automatically memorize 2x99 groups of measured value in memory and calculate the average value of latest three groups of measuring results. It will store correct user and measuring time if they are set correctly, otherwise, user and measuring time cannot be memorized correctly.

Please conduct initial setting before the first measurement if you want to use memory and average value function. Initial setting includes settings of user, time, volume and blood pressure unit.

3.1 User Setting

In the shutdown state, long press MEMORY key  until the user symbol " " on screen starts flashing up, long press MEMORY key  to switch " " or " " for choosing user, then short press MEMORY key  for confirming and switch to year setting mode

at the same time, if not to choose the system defaults user "1".


Long press "MEMORY" key for switch. Short press "MEMORY" key for confirming.

3.2 Time Setting

We can get known from 3.1, press MEMORY key  under user mode and it switches to year setting mode; the screen displays "2016" and flashes, long press MEMORY key, value increment can switch between 2016-2035, when the year to be set appears, short press MEMORY key  to confirm and it switches to "month", "date" setting mode at the same time, it means the value stands for month is in flashing state, long press MEMORY key, value increment can switch between 1-12, when the month to be set appears, short press MEMORY key  to confirm, the value stands for date is now in flashing state, it is the time to set date, long press MEMORY key, value increment can switch between 01-31, short press MEMORY key  to confirm and it switches to "hour", "minute" setting mode at the same time, value stands for hour is in flashing state now, long press MEMORY key , value increment switches between 0 to 23. When the hour to be set appears, short press MEMORY key  to confirm, the value stands for minute is now in flashing state, it is the time to set minute, long press MEMORY key, value increment can switch between 00-59, when minute to be set appears, short press MEMORY key  to confirm

and it switches to “sound” setting mode at the same time, namely “SP” setting mode.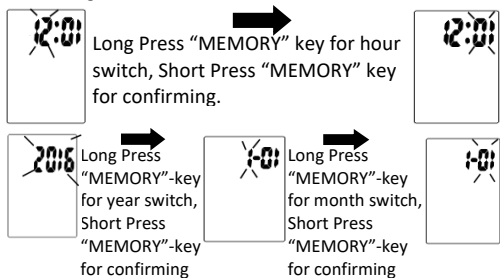

3.3 Sound setting

We can get known from 3.2, after setting up time, long press MEMORY key  to confirm and switch to "SP" mode at the same time, the "ON" or "OFF" is in a flashing state now. Long Press MEMORY key to choose, when set to "ON", there is a voice prompt when BP monitor is under measurement and viewing memory; when set to "OFF", there is no voice prompt. After choosing it, short press MEMORY key  for confirming and switch to “BP Unit” mode at the same time. The system defaults “ON” if you didn’t choose anything.


Long Press "MEMORY" key for switch, short Press "MEMORY" key for confirming.


Note: If the instrument is optional without voice, no need to set the sound.

3.4 BP Unit Setting

We can get known from 3.3, after setting up sound, press MEMORY key  to confirm and switch to "BP Unit" mode at the same time, the "mmHg" or "kPa" is in a flashing state now. the system defaults mmHg as blood pressure unit if there is no switch. After choosing it, short press MEMORY key  for confirming and it is shutdown at the same time.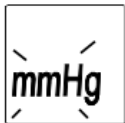

Press "MEMORY" key for switch
Press "SET" key for confirming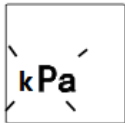

4. Usage of NIBP Cuff

- 1) Insert the cuff tracheal into the air tubes interface of blood pressure monitor.
- 2) Adjust the cuff to tubular until it is suitable to put your arm inside.
- 3) Arm through the cuff. The cuff air tube should be located the palm front end.
- 4) The cuff should be wind on the upper arm. ARTERY MARK

should be located in the middle of the inboard arm and pointing down to the inside of the arm, the air tube is also located in the inboard and should be in a line with middle finger.

5) The bottom of the cuff should be located on the elbow in upper arm, the bottom of the cuff should be 2cm-3cm from the elbow.


6) Please close the cuff by buckle. There should be not got the gap between the arms and the cuff.

5. Measure blood pressure

5.1 The Correct measurement method

The left and right arm are both ok to measure, be bare-armed or only wear thin clothes to measure. Please measure at a temperature suitable room.

- 1) Sit comfortably and naturally, with your feet flat on the floor, legs uncrossed
- 2) Put your elbows on the table;
- 3) The cuff height should be consistent with the heart;
- 4) The palm shall be upward, relax the body.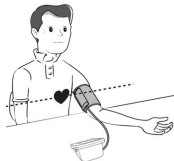

Attention

- If you wear thick clothes, don't roll up your sleeves, but take off and measure it.
- In order to accurately measure, please pay attention to the correct winding cuff (Refer to "4. Usage of NIBP Cuff")
- Please don't press your arms on the air tube, or you may restrict the flow of air to the cuff.
- Please be the same pose for blood pressure measurement, every day at the same time, with the same arm.


- The location of the cuff will cause changes of measurement results.
- Don't touch the device, cuff and air tube during measuring.
- You should keep quiet when start measuring, the body should be in a state of relaxation
- Please be quiet for 4~5 minutes before measurement.
- Relax your body, don't let muscle activity
- Don't talk or move when you are measuring.
- Please wait for 2-3 minute if you want to measure again.
- Do not use mobile phones and other mobile devices in the vicinity of the machine.

5.2 Operation steps of blood pressure measurement


The machine has 2 kinds of BP units, which is mmHg and kPa, please refer to the part for the specific method of setting.

Take the mmHg value as an example in the following explanation.

1) Press the "START / STOP" key, the machine will automatically return to zero, arm cuff will automatically start filling with air, the pump started to inflate the cuff, the screen will display the pressure changes in the cuff. In the process of inflation, arm type blood pressure monitor also detects the pulse. During the whole measurement process, please do not move the arm, keep the stationary state.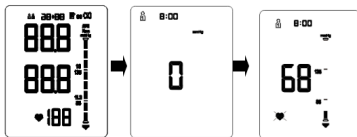

2) When the air in cuff reaches a stable pressure, the pump stops inflating, the pressure in the cuff gradually decreases and will be displayed on the screen at the same time, the heart icon will flash.

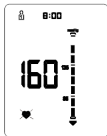3) After the measurement, the machine will simultaneously exhaust and displays your blood pressure value and the pulse number.

Note: the machine will automatically save the blood pressure value and the pulse rate.

Interrupt measurement:

If you need to interrupt the measurement for some reason (for example, you feel uncomfortable), at any time, press the "START / STOP" key, the machine will automatically reduce the air pressure, interrupt measurement.

Important hints:

For the same user, the interval between the two measurements is at least 2-3 minutes. Between the two measurements of waiting is mainly to let the artery restore to the state before the

measurement of blood pressure.

In the measurement process, if the body is moving, it will cause the measurement result to be inaccurate, or measure the failure, and it will display error message. In this case, please repeat the measurement process, and keep quiet until the end of the measurement.

If Irregular Heartbeat(IHB) brought by common arrhythmias is detected in the procedure of blood pressure measurement, a signal of

“” will be displayed. Under this condition, the blood pressure monitor can keep function, but the results may not be accurate, it's suggested that you consult with your physician for accurate assessment.

5.3 Answers to the question during measurement

Anomalies	Cause	Correction
A high (low) anomaly blood pressure measurement	The cuff is not properly wrapped	Wrap the cuff properly (refer to the part 5)
	Chatting or moving the arm during measurement	Keep quiet and body peaceful then measure again
	Sleeves rolled up too tight	Take the too tight coat off and measure again
No inflation	Cuff leakage	Change a new cuff
	Plug connected not well with the cuff	Please connect correctly
Cuff leak in a short time	Cuff wrapped too loose	Please wrap the cuff correctly
Not working after press the start button		Please reconnect power and measure again

During measurement suddenly power off	When not use for a long time, the temperature variation may make the battery power exhausted	Change a new battery
No display after press the start button	Power exhausted	Change a new battery
	Whether the polarity position anti-access	Install the battery in proper placement
Other anomalies	Please press the start button and check again or change a new battery, if still can't measure please connect After-sales service department.	

6.1.1 Read memory record

Short press the MEMORY key it will show the latest 3 times average value. It will appear "AVG" in the top-right corner;

Note

- When the number of memory value is less than 3 set, like there are 2 set memory value, the memory button will show this 2 set average value. If there is 1 set memory value, the memory button will show this 1 set average value, that is this 1 set measure value.

6.1.2 Read the result

Press the MEMORY key in average mode, you can read the latest 1-time measurement value, press the MEMORY button again to cycle to the previous measurements.

6.2 Delete the memory value

If you long press the MEMORY key in the memory mode, all memory values will be deleted.

Warning

- This operation deletes all memory values, users need to operate carefully.

7. Maintenance and Storage

Be sure to keep the notes and other correct usage in the user manual, Failure to obey the regulations, the company doesn't take responsibility of the quality.

7.1 Cleaning frequency

If the host is dirty when used at home, it is recommended to clean it. See 7.2 for cleaning method.

If the host is not dirty, simply clean the cuff before and after each use. It is recommended to put disposable tissue under the sleeve to reduce cross contact. See 7.2 for specific cleaning method.

7.2 Cleaning method


For home use: Using a soft dry cloth towel clean this arm type blood pressure monitor, if it is very filthy you can wet the towel with water or neutral detergent, wring out it and wipe the blood pressure monitor.

Smooth the cuff, wipe with 75% alcohol, and let dry.

When used in hospital, after use, the host and cuff should be wiped clean with 75% alcohol, dried and then disinfected by ultraviolet radiation.

Please prevent water or other liquid seep into the blood pressure monitor.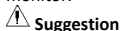

Don't clean the host and cuff with naphtha, thinner or gasoline etc. Do not get the cuff wet and do not attempt to clean the cuff with water.

Please prevent water or other liquid seep into the blood pressure monitor.

7.3 Storage method

Do not place the blood pressure monitor in the following places:

- Easy splashing
- Direct sunlight, high temperatures, humidity, dust, full of corrosive gas places
- Some slant, vibrative, shock place.
- Some place storing chemicals or corrosive gas.

**Advice**

Please take out the battery and keep it in well storage if you won't use the unit in long time. (More than 3 months), in addition into the battery is not used for a long time, may be due to battery leakage caused by fault;

**Advice**

- The height of the arm type blood pressure monitor should not exceed two meters.

**Advice**

- It is recommended to calibrate the arm type blood pressure monitor once a year, by the manufacturer.

7.4 Notes

- If the performance is inconsistent with the description or changes, stop using immediately and contact the manufacturer.
- Do not use this product if you are allergic to ABS and other materials.
- Direct use in the environment of electromagnetic interference shall be avoided to prevent temporary influence on its accuracy.
- The product is suitable for professional use or domestic use.
- The correct use method is the key to the accuracy of measurement. Operators should have certain knowledge of blood pressure. The operating environment shall be in accordance with the instructions.
- Too frequent measurements can cause injury to the patient due to blood flow interference.
- The effects of damaged sensors and electrodes or loosened electrodes, can affect performance or cause other problems.

- Please place the blood pressure monitor in a place where children, pets, etc. cannot be touched, so as to prevent falling, biting and affecting the product performance.
- Do not use beyond the service life of the product, otherwise the accuracy of the instrument will be affected
- The blood pressure monitor cannot be serviced and maintained during use.
- The blood pressure monitor should be avoided in places with poor ventilation and high dust and lint content. Avoid direct sunlight and strong light sources.
- If you need maintenance, please contact the manufacturer

7.5 Waste disposal

1. Please follow local laws to dispose of waste scrap.
2. Follow local ordinances and recycling instructions regarding to disposal or recycling of the device and device components, including used batteries and packaging box.

8. Manufacturer's Declaration of the EUT

Statement:

- The Arm type Blood Pressure Monitor or user should use the product in the electromagnetic environment specified in the following table, otherwise it may cause abnormal operation of the product.
- The internal structure of the Arm type Blood Pressure Monitor adds magnetic rings, magnetic beads, and conductive cloth to spray conductive paint to avoid electromagnetic interference, so as to prevent adverse events to patients and operators due to electromagnetic interference.
- Arm type Blood Pressure Monitor is table-top equipment

suitable for medical unit and home use.

- **Warning:** Use of this Arm type Blood Pressure Monitor adjacent to or stacked with other equipment should be avoided because it could result in improper observed to verify that they are operating normally.
- **Warning:** Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30cm (12 inches) to any part of the Arm type blood pressure monitor, including cables specified by the manufacturer. Otherwise, degradation of the performance of this arm type blood pressure monitor could result.

Novidion GmbH
Fuggerstr. 30, 51149 Cologne, Allemagne
Tel.: +49 (0) 2203 /9885200
Fax.: +49 (0) 2203 /9885206
www.pulox.de / info@pulox.de

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Novidion GmbH, Fuggerstr. 30, 51149 Köln, Germany
Xuzhou Yongkang Electronic Science Technology Co., Ltd
Prolinx GmbH
MedNet Swiss GmbH