EC-95

Ohrenschmalzentfernungs-Tool

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Dear Customer,

we are pleased that you have decided to purchase a product from our range. Please read these operating instructions carefully and carefully and keep them accessible for other users. Observe the application instructions and notes.

With kind regards
the Novidion team

Earwax Removal Tool EC-95

Instruction manual

1. Intended use

Excessive cerumen (earwax) can build up in the ear canal and cause blockages, hearing problems or tinnitus. Conventional cleaning (e.g. with conventional cotton swabs) can cause earwax to be pushed even deeper into the ear canal and harden.

The earwax removal tool EC-95 is

intended for preventive and acute detection and removal of earwax in persons aged 3 years and older and should only be operated by persons in full possession of their physical, sensory and mental faculties.

The product is suitable for use in private homes and medical facilities.

2. Contraindications

Do not use the product in case of:

- Inflammation in or around the ear
- Infections in or around the ear
- Increased irritability of the skin in the ear canal
- Advanced hardening of the earwax

3. Precautions

Please read the safety instructions carefully before using the equipment. Operators do not need professional training, but should use this product only after fully understanding the requirements in this manual. To avoid damage from improper use, use this product properly.

1. When cleaning the lens, please wipe

it carefully with a professional alcohol swab.

2. If the image is not clear, wipe the camera with a cotton swab.
3. When you move, please do not use the camera and protect it from being bumped by others.
4. This product is not suitable for children under 3 years.
5. Avoid direct sunlight on the product, especially on the silicone cover of the earpiece, to prevent softening.
6. Do not use the device in liquids to avoid damage due to immersion.
7. Avoid direct sunlight on the device, especially on the accessories, to prevent softening.
8. During use, the temperature of the product will rise slightly (up to 35°C), so please use it carefully.
9. To ensure an optimal user experience, please ensure that children maintain an upright sitting position during treatment and move as little

as possible.

10. Choking hazard: Please note that small parts contained may be swallowed and could lead to choking. Therefore, keep them out of the reach of children.
11. To keep the ear canal clean and safe, replace the silicone cover as soon as it becomes defective or has been used more than 30 times.
12. Press the key for 3 seconds to turn off the device.
13. Light Description: The red light is on when charging and turns off when fully charged. Slow flashing means there is no connection. Fast flashing means the device is out of power. Continuous blue light means that the connection is intact.
PS: Slow flashing and fast flashing refer to normal operation (not charging).
14. Wipe the lens with alcohol or water and do not allow the housing to come

into contact with water.

15. Do not disassemble, modify or repair this product yourself.
16. Should serious incidents occur, please report them immediately to the manufacturer as well as to the competent authority of the member state in which you are located.
17. if you experience symptoms such as severe pain in the ear, ear canal or eardrum, deterioration of hearing or tinnitus during or after treatment, discontinue use immediately and seek medical attention.
18. Dispose of biological waste and residues thereof securely packaged in a suitable container and do not leave in the open.

4. Product parameters

Product Name: Earwax Removal Tool

Model: Y13Pro, Notification number

DE/CA20/00192846

Weight: 18g

Network Standard: IEEE802.11b/g/n

Working frequency: 2.4 GHz

Frame rate: 20 Fps

Image sensor: CMOS

Operating temperature: 10~40°C

Battery: 230mAh

Input current: DC 5V 300mA

Optimal focal length: 1.5-2CM

Gravity sensor: Ultra HD camera

Device name: softish

LED: 6*LED

Lens waterproof: IP67

Classification: Medical Device Class I

5. Performance characteristics

- A. Easy to use and suitable for non-professionals.
- B. Rechargeable battery (charging cable type-C)
- C. Auto power off after 3 minutes without signal.
- D. Handy, light weight and mobile use.
- E. Image transfer to cell phone via bluetooth
- F. Integrated light source for better sighting of the ear.

6. Accessories

- Replacement silicone covers
- Type-C charging cable
- Ear care kit

7. Download the app

Search for the "EarVision" app in the App Store (Apple) or GooglePlay Store

(Android) and click on it to download it.

8. Connection procedure

Before using the product for the first time, make sure that the product has sufficient battery power.

Turn on the “Bluetooth” and “WIFI” functions on your cell phone.

Then press the power button (1)

for about 2 seconds to turn on the device.

Start the app. Grant the required authorizations the first time you start it.

When you open the app, your phone will ask you to allow “EarVision” to find and connect to devices on your local network. You must confirm this in order to use the app.

If you see a message on your phone when you open the app asking if you want the app to use Bluetooth, confirm

this.

If you see a message when you open the app asking if you want to allow EarVision to use wireless data, select “Wi-Fi only”.

If you see a message when you open the app asking if you want EarVision to join the Wi-Fi network “softish-xxxx”, select “Join”.

If you are asked whether you want to allow EarVision to receive location information, please select “exact location” and “only allow during use”.

Click on the “Search” icon in the middle of the screen.

A message may appear, such as: “WIFI can't connect to the internet, keep the wifi or switch to another?” (The message may vary depending on your phone). Please select “Keep wifi”.

Click to connect to the WIFI “softish-xxxx”.

9. Installing the ear swab

Before use, place the silicone cover vertically on the stick.

Note: The damper must be attached to ensure the device functions properly.

Note: Please remember that only the camera part is waterproof. The other device components are not waterproof.

10. Application

Use the app to take pictures and record videos.

Turn on the eardrum detection alert function as soon as you notice the eardrum detection icon.

1. Settings and options
2. Activation of the front camera
3. Lock alignment
4. Eardrum warning function
5. Adjusting the brightness

Meaning of the symbols:

Back

Photo button

Anti-Touch

Mirror mode

Record-Button

Image archive

Zoom

Settings

FAQ

Image archive

Privacy Policy

Search

11. Hazardous materials

Part name	(PB)	(Hg) (Ca)	(Cr(Vi))	(PBB)	(PBDE)
Printed circuit components	☐	☐	☐	☐	☐
Plastic case	☐	☐	☐	☐	☐
Plastic and rubber accessories	☐	☐	☐	☐	☐

☐ This form has been prepared in accordance with SJ/T11364.

It indicates that the content of harmful substances in this book is partially below the requirements specified in GB/T26572-2011.

12. Troubleshooting

Fault	Solution
The light does not turn on. No reaction after switching on.	1. Press the power button to turn on. 2. Please use the charging cable to charge to ensure sufficient power supply.
The app closes	Delete the existing APP and install it again
The ear stick does	Attach the silicone cover

not seem to be assembled correctly.	to the front end and make sure it is completely pulled over and fits correctly.
Image cannot be displayed. The image connection is stuck.	Exit the app and start it again.
The app cannot establish a connection with the product	Switch off the EC-95 and restart it after a few seconds.

13. Disposal

Dispose of the device in accordance with the Waste Electrical and Electronic Equipment Regulation 2002/96/EC - WEEE. If you have any queries, please contact the local authority responsible for disposal. The used, completely discharged batteries are to be disposed of via the specially marked collection containers, special waste collection points or via the electrical dealer. You are required by law to dispose of the batteries.

Note: You will find these signs on batteries containing harmful substances:

Pb = Battery contains lead

Cd = Battery contains cadmium

14. Symbols

Symbol	Meaning
	Attention! Please refer to the accompanying document (user manual)
IPX0	Protection class of the case
	Recyclable
	Batch number
	Keep dry
	Limitation of the storage temperature
	Manufacturer
	Class II devices
	This symbol indicates that WEEE cannot be disposed of as unclassified household waste, but must be

	recycled separately.
	Medical device
	Attention. Please refer to the accompanying document (user manual)
EMC	Electromagnetic compatibility
P/N	Manufacturer material code
	Expiration date
	Limitation of the atmospheric storage pressure
	Limitation of humidity during storage
	Date of manufacture
SN	Serial number
	This product complies with Directive (EU) 2017/745 on medical devices, a directive of the European Economic Community
	Applied part type BF
	European representative

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Novidion GmbH
Shenzhen Jiding Electronic Co., Ltd.
Humiss Beratung GmbH